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
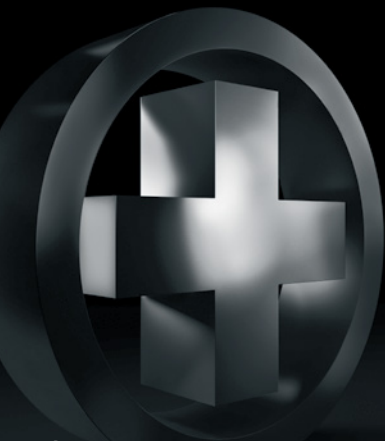
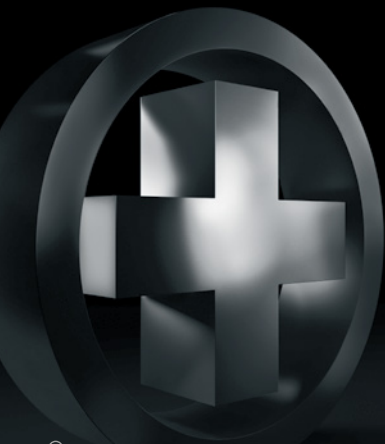
The first choice for most demanding people.

select[®] - our highest-quality series of supports convinces for its functionality and the excellent product components. It offers unique characteristics, which satisfy even the most demanding patients.

select[®]

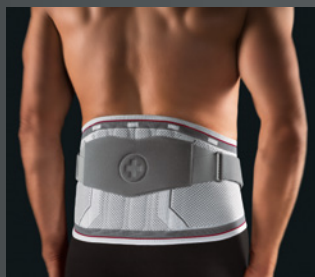BORT. Benefit at your side.

EN Dear customer,

we are happy that you have selected a high-quality medical product from BORT GmbH. With the **BORT select® Lumbar Spine Brace for Mobilisation** you have received a product from your medical specialist dealer which meets the highest medical, functional and qualitative requirements.

The **BORT select® Lumbar Spine Brace for Mobilisation** supports damaged tissue structures. The brace compression supports the retaining function of the ligaments, thus relieving the vertebrae.

The removable restraint achieves additional stabilisation and straightening of the lumbar spine. By using it, lumbar spine relief is improved and can be dosed without graduation and individually. The removable restraint enables adaptation of the stabilising effect to the course of therapy.

INSTRUCTIONS FOR USE

Place the **BORT select® Lumbar Spine Brace for Mobilisation** around the waist for mobilisation and close the front fastener, using the hand straps as supports.

Pay attention to comfortable fitting. In order to additionally relieve the lumbar spine, tighten the side restraints evenly and simultaneously.

Observe simultaneous pull on both side restraints in order to avoid undesired rotation on the spine.

During the course of therapy, the restraints can be removed individually. For this, please refer to your treating physician. In this case, please release the Velcro fastener directly centrally in the back area and respectively at both abdominal latch plates.

INDICATIONS

Moderate lumboischialgia with mild deficits with herniated discs/prolapse, moderate radicular, pseudo-radicular lumbar syndrome, grade I spondylolisthesis with low back pain, moderate lumbar deformity (recurrent treatment, complicated history), i.e. after inter-vertebral disc surgery with mild to moderate prolapse.

CONTRAINDICATIONS

Lymphatic drainage disorders including unclear soft tissue swelling, sensory disturbances and circulatory disorders in the specific body region, skin diseases in the specific part of the body, latex allergy.

COMPOSITION

68 % Elastodics

32 % Polyamide

IMPORTANT NOTE

No reutilisation – this medical device is meant for the care of one patient only.

The **BORT select® Lumbar Spine Brace for Mobilisation** supports the healing process especially during activity. If instructed to wear the support at night, you must take precautions to avoid impairment of blood circulation. If you experience numbness, please loosen the support immediately and, if necessary, remove it.

If you suffer from persistent complaints or if you experience worsening complaints, you should consult with your doctor.

Whilst wearing the brace, please neither use any creams nor ointments as they can destroy the material.

The product contains latex and can trigger allergic reactions.

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BORT GmbH | Postfach 1330 | D-71367 Weinstadt | www.bort.com