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EN

Back Supports support injured tissue structures (bones, ligaments and cartilages). Hereby the compression effect supports the holding function of the local ligament and muscular function, and thus relieves the vertebral column.

Instructions for use:

Close the back support in the abdominal area using Velcro fastening. The support should sit tightly, but not constrain.

The elastic additional straps, which are also fastened on the stomach side using Velcro, regulate compression and the corresponding stabilisation of the lumbar spine.

Indications:

Treatment of increased pain in the lumbar spine, severe low back pain, recurrent low back pain, osteochondrosis, spondylarthrosis, mild lumboischialgia, mild radicular syndrome, spondylolysis without spondylolisthesis, mild to moderate lumbar deformity (degenerative spondylolisthesis), after inter-vertebral disc surgery.

Contraindications:

Lymphatic drainage disorders including unclear soft tissue swelling, sensory disturbances and circulatory disorders in the specific body region, skin diseases in the specific part of the body.

Composition:

76 % polyamide, 13 % cotton, 11 % spandex

Important Note:

No reutilisation – this medical device is meant for the care of one patient only. In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

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