BORT AktiVen® 100

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Fitting instructions

For compression stockings with closed toe

1 Pull the stocking over the lower arm and take hold of the inside of the heel.

2 Hold the heel firmly, pull the stockings forwards and turn them inside out up to the heel.

Tip: Use the BORT AktiVen® Special Gloves for the following steps.

3 Take hold of the overlapping material in the heel area and slip over your foot until the knitted heel fabric is over the heel.

Tip: For the stockings, carry out steps 1–5 first with one leg and then repeat for the second leg.

4 Pull the stockings up over the heel gradually and thus simultaneously back with the right side out.

5 Then slip the stockings up step by step towards the knee, using the palm of your hand to shape them to the leg. At the same time, never pull the upper edge!

Tip: Turn the gripping edge (if one exists) inside out before putting them on.

6 Then repeat steps 1 to 4 for the stockings with the second leg. Turn both sides back the right way out and slip them up both legs. Then, adjust the positioning of the fabric with the palm of your hand.

7 Finally pull the body belt up to the waist whilst observing that the gusset is vertical in the crotch.

Attention: Your compression stockings must never be twisted. Put the stockings gradually in place from the bottom to the top.

These tips make them easier to put on:

- Put the compression stockings on in the morning, directly after getting up.
- Rough, unfiled fingernails, sharp-edged rings and toenails, cornea and faulty shoe lining can damage the knitted fabric.
- Please always use BORT AktiVen® Special Gloves for putting on the compression stockings when getting dressed.

BORT AktiVen® Medical compression stockings

Dear customer,

we are happy that you have selected a high-quality medical product from BORT. With BORT AktiVen® Medical compression stockings, you have selected a product from your specialist dealer which meets the highest medical, functional and qualitative requirements.

The pressure from the BORT AktiVen® Medical compression stockings narrows the vein diameter. The venous valves can close again, thus supporting venous reflux from the legs. As a result of the narrower vein diameter, blood circulates faster again, preventing the formation of blood clots or thrombosis. The BORT AktiVen® Medical compression stockings fittingly selected by the specialist dealer provide immediate noticeable relief, enabling the slowing down of the chronic progress of the illness or even hindering it.

Material composition

The sewn in textile label provides the precise material composition.

Instructions for washing and care

Before washing, all fasteners (if existing) must be closed! Please always wash stockings with grip top and lace grip top inside-out. Wash the compression stockings separately the first time (the dye may run). Your compression stockings should be hand-washed or washed using a delicate wash cycle (40 °C). We recommend BORT Special Detergent for BORT AktiVen® Medical compression stockings.

Tip: A laundry net provides additional protection for the knitted fabric. If hand-washing, rinse thoroughly, do not wring. You can reduce the drying time by placing the compression stockings on a thick terry towel after washing them, rolling them up and pressing out excess moisture firmly. Then, hang the stockings up loosely. Do not leave them around in the towel and neither dry them on a radiator nor in the sun. The compression stockings must not be cleaned chemically. Please observe the care instructions on the textile label sewn in.

 Delicate wash

 Do not iron

 Do not bleach

 Do not clean chemically

Please wash inside-out! No fabric conditioners!

Indications

Varicose

- Primary and secondary varicose
- Varicose veins during pregnancy
- Supporting sclerotherapy
- Following surgical vein operations

Thromboembolism

- Thrombophlebitis (superficial) as well as condition after healed phlebitis
- Deep leg vein thrombosis
- Condition after thrombosis
- Postthrombotic syndrome
- Thrombosis prophylaxis for mobile patients

Chronic venous insufficiency (CVI)

- Stages I–III CVI according to Widmer or C₁–C₆ according to CEAP

- Ulcer prevention
- Deep vein insufficiency
- Angiodysplasia

Oedemas

- Lymphoedemas
- Oedemas during pregnancy
- Posttraumatic oedemas
- Postoperative oedemas
- Cyclically idiopathic oedemas
- Lipoedemas from stage II
- Lip lymphoedemas
- Phlebo lymphoedemas
- Congestion as a result of immobility (arthrogenic congestion syndrome, pareses and partial pareses of the extremity)

Further indications

- Post-burn condition
- Scar treatment

Contraindications and risks

Absolute contraindications

- Advanced peripheral arterial occlusive disease
- Decompensated heart failure
- Septic phlebitis
- Phlegmasia coerulea dolens

Relative contraindications

- Distinct weeping dermatoses
- Intolerance of compression stocking material
- Severe paraesthesia of the extremities
- Advanced peripheral neuropathy (e.g. diabetes mellitus)
- Primary chronic polyarthritis

Side effects

None known if used correctly.

Should nonetheless negative changes (e.g. skin irritation) occur during the prescribed administration, please contact your doctor or specialist medical retailer without delay.

Should you be aware of an intolerance against one or more of the constituents of this product, please confer with your doctor before using it. The manufacturer is neither liable for damagers/injuries arising from incorrect handling or incorrect use.

Please observe:

- Only wear your compression stockings if prescribed by a doctor. The prescription should only be handed over to you by trained medical retail trade staff.
- Continuation of therapeutic success or changed ailments make it necessary to take new measurements and, if applicable, to re-modelling.
- Your compression stockings are to a great extent resistant against oil, ointment, skin moisture and further environmental influences. Certain combinations can influence the durability of the knitted fabric. We therefore recommend regular checking by the medical specialist retail trade.
- Please observe the following for compression stockings with a grip top: the legs should be depilated and free of oil, powder, creams and ointments.
- In case of damages, please refer to your specialist retailer. Do not repair yourself – this may influence quality and medical effectiveness.

Storage information

- Store dry at room temperature and protect against direct sunlight.
- Series products have a maximum shelf life of up to 36 months, which is marked with a sandglass symbol on the packaging label. After this, the period of use for the compression stockings is a maximum of 6 months.
- Due to material attrition or fatigue, medical effectiveness can be guaranteed for a maximum wearing period of 6 months for series sizes and customised products. This assumes correct handling (e. g. care and when putting them on and taking them off).

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BORT Special Adhesive Lotion

- *For the safe fixing of BORT AktiVen® Medical compression stockings*
- *Skin-friendly, can be removed from the skin without any traces using water*

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BORT Special Detergent

- *Specifically for knitted medical compression fabrics*
- *Top purity when hand-washing*

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BORT AktiVen® Special Gloves

- *Simplifies putting on and taking off medical compression stockings*
- *Very good adhesion for perfect distribution and positioning of the knitted fabric*

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BORT AktiVen® Donning Aid for medical compression stockings

- *Enables independent and effortless putting on of medical compression stockings*
- *Above all for patients with limited strength and restricted mobility*

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For daytime use – BORT WELEDA Venadoron

- *Cares and invigorates your legs*
- *Ideal before putting your compression stockings on*

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For nighttime use – BORT WELEDA Sea Buckthorn Body Lotion

- *Nourishes dry skin, lastingly preserves its moisture content*
- *Ideal after wearing your compression stockings*

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**products available in Germany and Austria only*

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BORT GmbH