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EN

Relief and support is a basic principle when treating injuries and excess stress as they considerably accelerate the healing process and prevent resulting health damages. Ankle joint braces support damaged tissue structures (joints, ligaments, capsules and cartilage), stabilise, guide and thus contribute to the positive healing process.

Instructions for use:

Important: The BORT TaloXpress is worn over the socks, not directly on the skin.

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Putting the brace on:

Open all the brace's fasteners. For this purpose, unloop the belt straps. If required, the strap length can be set by shortening the straps.

Place your foot into the brace, here the plastic shell must be positioned on the inner side of the leg. The lower leg should be positioned vertical to the ground.

1. Close the fastener on the metatarsus first, it is marked with a "1".
2. Then, fix the fastener on the upper brace edge which is marked with the number "2".
3. In the next step, close fastener "3" on the lower, blue Velcro surface

Finally, fastener "4" is used to set the stabilisation degree. For this purpose, guide the sealing tape diagonally over the instep and fix it to the upper blue Velcro surface under slight tension. To improve relief, the outer edge of the foot can be lifted slightly whilst the diagonal restraints are positioned.

If required, the locking belts can be readjusted after the brace has been positioned.

Indications:

Conservative treatment of ankle distortions and ligament ruptures, post-operative protection after ligament suture/reconstruction, immobilisation in the case of inflammatory irritations, rheumatism, arthritis, tendinitis, supination/pronation trauma.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymph drainage disorders, also unclear soft tissue swellings distal to the aid positioned, sensory and circulatory disorders with the region of the body treated, skin diseases in the part of the body treated.

Material composition:

49 % polyamide, 30 % microfibre, 14 % polyester, 7 % polyurethane

Important Note:

No re-use – this medical aid is intended for treating one patient. If wearing at night is prescribed, avoid negative influence on the circulatory system. In case of numbness, loosen the bandage and remove if necessary. In case of persistent complaints, consult the doctor. Whilst wearing the support, please neither use any creams nor ointments as they can destroy the material.

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