IT

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EN

Bort AchilloStabil® Ankle Support offers relief of achilles tendon by means of integrated pads.

When attaching the support care should be taken that the two semi-lunar pads are located in the area between achilles tendon and inner resp. outer ankle.

For additional relief of the achilles tendon one of the attached heel cushion can be put into the shoe. To avoid differences in leg lengths between affected and non-affected side, put the second attached heel cushion into the shoe of the non-affected leg.

Indications:

Chronic or post-traumatic irritation or tenderness in the Achilles tendon area, Haglund syndrome heel irritation, achillodynia, irritation and pain of the Achilles tendon and surrounding areas, after surgery, for example in the case of Achilles tendon rupture, bursitis subachillea.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

Composition:

57 % polyamide, 28 % elastodien, 8 % spandex, 7 % viscose

Important Note:

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

While wearing the support do not use creams or ointments. This can damage the material.

The product contains latex and can trigger allergic reactions.

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