IT

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**EN**

Elbow Supports support injured tissue structures (joint, ligaments, capsules, cartilages). The compression hereby supports the holding function of the ligaments and is thus relieving the joint. At the same time the support helps to a high degree to avoid wrong movements. The wide area silicone pad protects the total pressure-sensitive elbow area.

**Indications:**

Chronic, post-traumatic or post-operative irritations of soft tissues in the area of the elbow joint, for example in the case of compression of the nervus ulnaris at elbow (sulcus ulnaris syndrome/cubital tunnel syndrome), bursitis.

**Contraindications:**

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

**Composition:**

57 % polyamide, 28 % elastodien, 8 % spandex, 7 % viscose

**Important Note:**

No reutilisation – this medical device is meant for the care of one patient only.  
In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.  
While wearing the support do not use creams or ointments. This can damage the material.

The product contains latex and can trigger allergic reactions.

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