IT

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Hand-wash the support in lukewarm water using a mild detergent. Do not use fabric softeners. Shape by hand and allow dry in the open air. Avoid heat.

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**EN**

Elbow Supports support injured tissue structures (joint, ligaments, capsules, cartilages). The compression hereby supports the holding function of the ligaments and is thus relieving the joint. At the same time the support helps to a high degree to avoid wrong movements.

**Indications:**

Chronic, post-traumatic or post-operative irritations of soft tissues in the area of elbow joint, i.e. in the case of articular effusion, sprains, arthrosis, contusions.

**Contraindications:**

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

**Composition:**

75 % polyamide, 25 % spandex

**Important Note:**

No reutilisation – this medical device is meant for the care of one patient only.

Wear support only when under strain, not during rest or sleep. Avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

While wearing the support do not use creams or ointments. This can damage the material.

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