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EN

Immobilisation is a basic principle during treatment of injuries and overstress, as the recovery process is considerably accelerated.

Instructions for use:

This is an one-piece brace and comprises a freely positionable elbow pad, which is Velcro-fastened in the brace. Prior to putting-on the BORT KubiFX, the two pre-formed alu-splints, which are placed in the splint pockets at the rear of the brace, have to be re-adjusted in accordance with the desired bent position, in which the elbow joint shall be fixed. The integrated pre-formed alu-splint at the volar side of the arm can be readjusted as well, if required.

Subsequently the elbow pad has to be correctly positioned and the fixing straps, which are provided with a slip-out-lock, are to be led through the D-rings.

Now the brace can be put onto the arm; the elbow tip has to be placed onto the pad in the curve (curvature) of the brace.

The first Velcro fastener has to be led over the soft parts between thumb and forefinger and Velcro closed at the brace. Both fixing straps, already led through the D-rings, are to be reversed, starting at the forearm and Velcro-closed. It is essential to pay attention that the elbow joint is indeed immobilised, however no congestion of blood circulation will occur.

The reversing straps provided at the inner and outer side in the region of the elbow bend serve as an additional hindrance against straightening and will be adjusted individually.

Indications:

Pre-operative, post-operative, post-traumatic, for example in the case of proximal sulcus ulnaris syndrome, descending tendopathies, epicondylitis, condition after bursectomy.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

40 % PUR foam, 30 % cotton, 30 % polyamide

Important Note:

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear brace during night, avoid hindering blood circulation. In case of numbness loosen brace and remove if necessary. If complaints continue, please contact the doctor.

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