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**EN**

Immobilisation is a basic principle during treatment of injuries and overstress, as the recovery process is considerably accelerated.

### **Instructions for use:**

Before putting the BORT KubiFX Long Light Elbow Brace on for the first time, both internal aluminium splints must be formed by an expert in accordance with the desired joint position. Then the two clamp screws must be loosened with the wrench provided and the brace length on the lower arm set to the patient's individual lower arm length by shifting it. Retighten the screws and check the sturdiness of the splint-clamp connection. Following this, direct the straps through the guide loops up to the anti-slip lock and fit the brace to the arm: the tip of the elbow (olecranon) must lie in the aluminium splints' integrated bend (curvature) and the hand securely grasp the handle. The brace's fixing straps are then closed with the desired tension. Both middle belt straps cross distally to the elbow bends in the lower arm region, thus not causing any undesired pressure on the nerves and blood vessels located there. Finally, it must be checked that, despite secure fixing of the elbow joint, no blood flow congestion is incurred.

### **Indications:**

Pre-operative, post-operative, post-traumatic, for example in the case of proximal sulcus ulnaris syndrome, insertional tendinopathy, pepicondylitis, post-bursectomy conditions.

### **Contraindications:**

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

### **Composition:**

40 % PUR foam, 30 % cotton, 30 % polyamide

### **Important Note:**

No reutilisation – this medical device is meant for the care of one patient only.  
In case of doctor's order to wear brace during night, avoid hindering blood circulation. In case of numbness loosen brace and remove if necessary. If complaints continue, please contact the doctor.

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