BORT Ellenbogen- ROM-Orthese

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Med
Mitten im
gesunden
Leben

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



REF 055 500

Illustrationen
Figures




1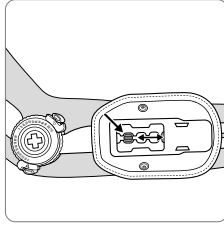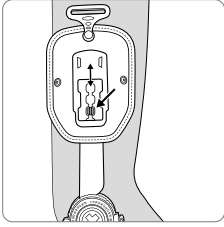



2


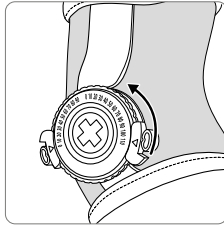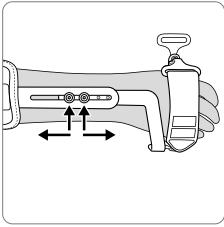
3

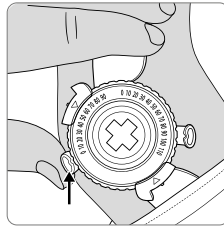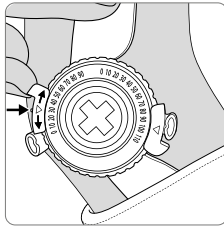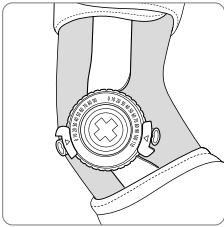4

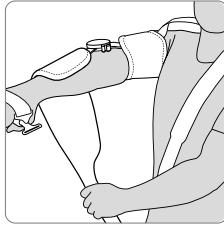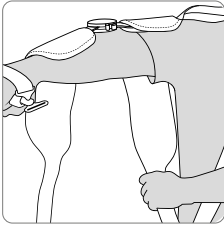5

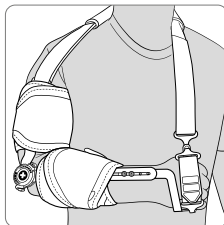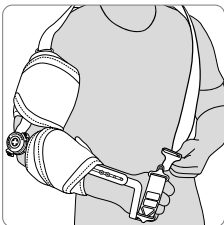Sprachen Languages

DE	Deutsch	Gebrauchsanweisung	04
EN	English	Instructions for use	07
FR	Français	Mode d'emploi	10
ES	Español	Instrucciones de uso	13
IT	Italiano	Istruzioni per l'uso	16
NL	Nederlands	Gebruiksaanwijzing	19
CS	Čeština	Návod k použití	22
ET	Eesti	Kasutusjuhend	25
PL	Polski	Instrukcja użytkowania	28
RO	Română	Instrucțiuni de utilizare	31

PDF: ga.bort.com

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Many thanks for placing your trust in a medical device from BORT GmbH. Please read the existing instructions for use carefully. If you have any questions, please refer to your physician or the specialist retailer from whom you purchased this medical device.

Intended purpose

This medical device is a brace for mobilisation of the elbow joint with adjustable scopes of movement and immobilisation of the proximal radius ulnar joint using fixing in the metacarpophalangeal joint.

Indications

Early functional postoperative or posttraumatic mobilisation with restricted movement of the elbow joint and immobilisation of the proximal radial ulnar joint, e.g. conservative/postoperative care, from radial head fractures, elbow luxation and distal humerus fractures to prosthesis implantation.

Contraindications

Peripheral arterial occlusive disease (PAOD), lymph drainage disorders, also unclear soft tissue swellings distal to the aid positioned, sensory and circulatory disorders with the region of the body treated, skin diseases in the part of the body treated.

Application risks/Important notes 

This medical device is a prescribable product. Discuss the use and duration with your treating physician. The expert staff from whom you have received the medical device will select the appropriate size and instruct you regarding its use.

- remove the medical device prior to radiological examinations
- do not close it too tightly during longer periods of rest, e.g. when sleeping
- in case of numbness, loosen the medical device and remove it if necessary
- in case of persistent complaints, consult the physician or specialist retailer
- use the medical device in accordance with therapeutic needs
- only use other products simultaneously after consultation with your physician
- do not make any changes to the product
- do not wear it on open wounds
- do not use in case of intolerance of one of the materials used
- no re-use. This medical aid is intended for treating one patient only
- whilst wearing the support, please neither use any creams nor ointments on or around it as they can destroy the material

Fitting instructions

1 In order to maintain correct functioning of the brace, the upper and lower arm cuffs must be fully opened or removed in a first step. Following this, the optimum positioning of the upper and lower arm cuffs is determined. Press the lock on the plastic support downwards and keep it pressed, setting can be made by sliding the module support distally or proximally. These processes should take place directly

with the patient. Then, let go of the locking bar and let it engage. Adapt the removed upper and lower arm cuffs again on the support modules.

2 The next step is the determination of the handle length. For this purpose, the Allen screws on the aluminium splint are opened and the handle is positioned so that the brace joint is precisely at the level of the elbow pivot point. If a lower arm is shorter or longer, the Allen screws can be positioned further proximally or distally, using the additional thread holes in accordance with the arm length.

3 Before the aid is positioned on the patient, the scope of movement (extension and flexion) is set. For this, the L-shaped locking blocks are opened clockwise with the finger. Then, the adjustment pins can be individually positioned with the desired value (extension/flexion) by pushing them in. To fix the scope of movement set, push the locking blocks back again anticlockwise and let them engage.

4 When then positioning the aid, the patient reaches for the handle with hand from the arm affected. Initially, the patient takes hold of the upper cuff with the hand from the side not affected and wraps it around the upper arm, creating pleasant wearing comfort and simultaneous tight fitting. The micro Velcro provided is used here to fix the cuff. Then, the lower arm cuff is closed using the same procedure.

Information: The cuffs should sit tightly on the upper and lower arm, but not cut off the blood circulation.

5 A shoulder strap is adapted for full immobilisation of the elbow joint. It is threaded in the upper area using a strap ring. For adaptation, it is placed over the shoulder and also threaded on the handle with a strap ring.

Finally, the correct positioning of the brace must be checked.

Removal

Place the shoulder strap aside. Then, open the lower arm cuff first, followed by the upper arm cuff.

Material composition

Polyamide (PA), polyurethane (PUR), polypropylene (PP), elastane (EL), cotton (CO), polyester (PES)
The sewn in textile label provides the precise material composition.

Cleaning information

 Hand-washing 30 °C  Do not bleach  Do not dry in a tumble dryer  Do not iron

 Do not clean chemically

Do not use fabric conditioner. You can remove the Velcro attachment of the lower and upper arm cuffs for cleaning purposes. Ensure that all the Velcro fasteners are closed prior to washing in order to avoid damaging other items of clothing. Stretch back into shape and dry in the fresh air. Avoid heat.

Guarantee

The legal regulations of the country in which you acquired the product apply to the purchased product. Please contact your specialist retailer if you suspect a warranty claim. Please clean the product before submitting a warranty claim. If the enclosed instructions for use have not been properly observed, the warranty may be impaired or cancelled. The warranty does not cover use of the product inappropriate for the indication, non-observance of application risks, instructions and unauthorised modifications to the product.

Useful life/Lifetime of the product

The lifetime of the medical device is determined by its natural wear and tear if treated appropriately and as recommended.

Duty of notification

If a serious worsening of the state of health occurs when using the medical device, you are obliged to notify the specialist dealer or us as the manufacturer and the Medicines & Healthcare products Regulatory Agency (MHRA).

You can find our contact information in these instructions for use. You can find the contact information for the national authority responsible for your country under the following link: www.bort.com/md-eu-contact.

Disposal

Upon the termination of use, the product must be disposed of in accordance with the corresponding local requirements.

Declaration of conformity

We confirm that this device conforms with the requirements of REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL. You can find the current declaration of conformity under the following link: www.bort.com/conformity

Status: 02.2024

MD Medical device |  Single patient – multiple use

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