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

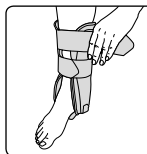
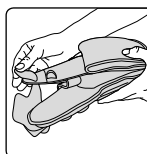
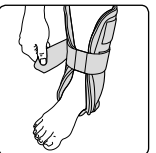
EN**Instructions for use**

When attaching the brace, care has to be taken that the shells are laterally positioned exactly centrally at the shank.

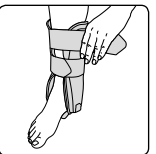
Please set the strap for the sole of the foot to fit the patient's circumstances in order to achieve effective stabilisation. Lay the wide lower strap circumferentially around the shells and attach it with Velcro.

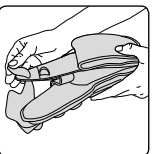Lay the upper strap around the shells and attach it with Velcro. If required, adjust the lower strap so that the brace is firmly in position.

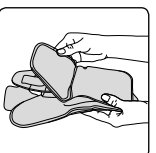If the tumefaction decreases, the MalleoStabil® Brace might loosen. Readjust if required.

**Use of gel component**

Pull off gel component from below upwards. When reattaching the gel component take care that it will be located exactly at the original place. The correct positioning assures that the plastic edges are covered.

For cryotherapy

- Remove gel component and put it for one hour into the deep-freeze. Note: gel will not become solid.
- Reattach gel component at the original place.
- Get refrigerated gel component not directly in touch with skin.
- For maximum wearing comfort always wear socks.

Foam component

The gel cushions may be replaced by the attached foam cushions if necessary – concerning this please absolutely take care of the doctor's instructions.

Use of foam component

Pull off foam component from below upwards. When reattaching the foamed component take care that it will be located exactly at the original place. The correct positioning assures that the plastic edges are covered.

Indications:

Post-operative protection after ligament suture, conservative treatment of ankle distortions of any severity, functional treatment of ligament ruptures, ankle fractures (Weber A), chronic ligament instability, shortening of time spent in plaster, ligament injuries, pronation and / or supination trauma, fibular ligament rupture.

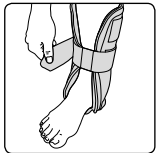
Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition: 80 % polypropylene, 20 % polyamide velour

Important Note:

No reutilisation – this medical device is meant for the care of one patient only. In case of doctor's order to wear brace during night, avoid hindering blood circulation. In case of numbness loosen brace and remove it if necessary. If complaints continue, please contact the doctor. CAUTION: If the gel components should leak and the gel comes into contact with the skin, wash the area immediately with soap and water and rinse well. In case of eye contact, rinse the eye under running water for several minutes holding the eyelid wide open. After swallowing, rinse the mouth and drink plenty of water. Consult a doctor if feeling unwell.

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