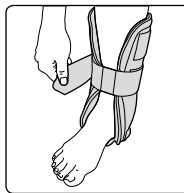
ES

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

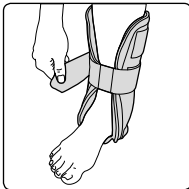
EN**User instructions:**

When attaching the brace, care has to be taken that the shells are laterally positioned exactly centrally at the shank.

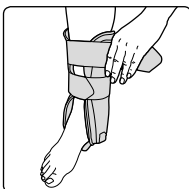
Please set the strap for the sole of the foot to fit the patient's circumstances in order to achieve effective stabilisation. Lay the wide lower strap circumferentially around the shells and attach it with Velcro.

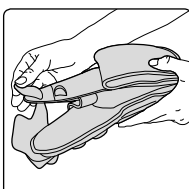Lay the upper strap around the shells and attach it with Velcro. If required, adjust the lower strap so that the brace is firmly in position.

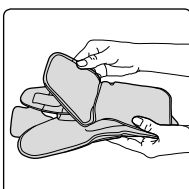If he tumefaction decreases, the Malleo-Stabil®-SOFT Brace might loosen. Readjust if required.

**Use of foam component:**

Pull off foam component from below upwards. When reattaching the foamed component take care that it will be located exactly at the original place. The correct positioning assures that the plastic edges are covered.

Indications:

Post-operative protection after ligament suture, conservative treatment of ankle distortions of any severity, functional treatment of ligament ruptures, ankle fractures (Weber A), chronic ligament instability, shortening of time spent in plaster, ligament injuries, pronation and / or supination trauma, fibular ligament rupture.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

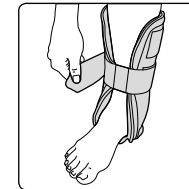
Composition:

80% polypropylene, 20% polyamide velour

Important Note:

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear brace during night, avoid hindering blood circulation. In case of numbness loosen brace and remove it if necessary. If complaints continue, please contact the doctor.

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