IT

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EN

Wrist Supports give relief in case of injured tissue structures (joint, ligaments, capsules, cartilages).

The BORT ManuCarpal®-SOFT Brace holds the fingers, thumb and the wrist in a resting position. The inserted aluminium splint enables individual adjustment in thumb and wrist area.

Additionally the splint offers increased protection against wrong movements.

Instructions for use:

- Put brace onto the volar side and decide on desired thumb and hand position. Take off brace and bend it manually into the defined shape.
- Put on brace again and first fix it with the Velcro strap over metacarpal joint, then close the straps around the wrist and lower arm.

Indications:

Pre-operative, post-operative, post-traumatic conditions, carpal tunnel syndrome (CTS), paralysis, inflammatory and rheumatic diseases.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

38 % polyamide velour, 28 % cotton, 28 % polyurethane, 6 % polyester

Important Note:

No reutilisation – this medical device is meant for the care of one patient only. In case of doctor's order to wear brace during night, avoid hindering blood circulation. In case of numbness loosen brace and remove it if necessary. If complaints continue, please contact the doctor.

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