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EN

Immobilisation is a basic principle for treatment of injuries and strains, as thus the recovery process will be considerably accelerated.

Wrist Supports support injured tissue structures (joints, ligaments, capsules, cartilages).

The BORT ManuDisc® acc. to Dr. Schütz results in immobilisation of the hand by the volar integrated alu-splint in combination with the circular embracing support.

Caused by the conception of the alu-splint an almost complete immobilisation of the wrist can be achieved in the sense of the restriction of dorsal extension, dorsal flexion, ulnar deviation in the wrist as well as the pronation and supination. The mobility of the fingers is not restricted.

Instructions for use:

Before putting on the BORT ManuDisc®, it will be necessary to have a health professional shape the aluminum splint to match the contour of your hand. This is best achieved by first bending the volar splint at the wrist to match the desired position of the hand. Now shape the stabilizing segments located above the wrist to match the contour of the lower arm. Carefully shape the grip on the outside of the metacarpus to match the contour of the hand. Then fasten the Velcro straps at the lower arm and the hand, making sure while doing so that the straps do not cut off the flow of blood.

Indications:

Pre-operative, post-operative, post-traumatic conditions, carpal tunnel syndrome (CTS), disc injury, carpal arthrosis.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

83 % polyester, 17 % polyester microfibre

Important Note:

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove if necessary. If complaints continue, please contact the doctor.

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