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EN

Wrist Supports support injured tissue structures (joint, ligaments, capsules, cartilages). The compression hereby supports the holding function of the ligaments and is thus relieving the joint. The aluminium splint inlay is formed in a way that the hand is hold in a relaxed, easy position. Additionally the splint offers increased protection against wrong movements.

Instructions for use:

The BORT ManuCarpal® Combi stabilizes the wrist and fingers (D. II - D.V). The finger rest can be conveniently removed and reattached (e.g. when wearing the aid during the day) using the Velcro strap and the circular band. The inner adjustable palm cushion provides additional wrist comfort and protects against pressure irritation.

When putting on the brace for the first time, start by bending the aluminum splint into the desired shape. Open all of the Velcro straps and place your hand into the brace. Close the Velcro straps along the wrist in a proximal to distal sequence. Then position your fingers on the finger rest and fasten the Velcro strap.

Indications:

Pre-operative, post-operative, post-traumatic conditions, degenerative, e. g. carpal tunnel syndrome (CTS), tenosynovitis, peripheral nerve paralysis.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

Composition:

86 % polyester, 14 % polyamide-silk

Important Note:

No reutilisation – this medical device is meant for the care of one patient only. In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

The product contains latex and can trigger allergic reactions.

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