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## EN

Braces in the hand region support damaged tissue structures (joints, capsules, ligaments and cartilages).

The brace supports and relieves the wrist, the metacarpus and fingers.

The immobilisation of a treated wound is important for undisturbed wound healing. This is primarily true for wounds close to joints.

All or individual long fingers can be fixated or exercised in the splint as early functional therapy. The pre-forming of the brace incorporates a metacarpophalangeal flexion of 70 – 80° and with a wrist stretching of 25°. (At night or in case of corresponding indication, the middle and end finger joints can be held stretched with Velcro tape.) Immobilisation in the protected or Intrinsic Plus position tenses the metacarpophalangeal collateral ligaments, thus avoiding contractures.

### Instructions for use:

**Important:** The BORT Dorsal Hand Brace may only be adapted by trained expert staff! When delivered, the brace is anatomically pre-formed. If required, let the brace reshape or remodel as instructed by the specialist!

Before putting the splint on, open all the Velcro straps. Place the splint on the back of the hand and close the Velcro fastenings around the wrist and forearm. Then close the Velcro fastening which fixes the metacarpus and dorsal brace together. Finally, the circular Velcro fastener which fixes the fingers to the dorsal brace is closed. Any protruding Velcro tape can be cut off. The integrated aluminium splint can be adapted manually (is cold formable) and can be removed from the pad cover if required.

### Indications:

Pre-operative, post-operative, post-traumatic, metacarpus and proximal finger phalanx in the area of the long fingers (e.g. a boxer fracture), injuries and diseases which make the immobilisation of long fingers necessary (for example infections)

### Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, Tendon injuries, CRPC (Morbus Sudeck), burns, compartment syndrome.

### Composition:

50 % PUR foam, 50 % polyamide

### Important Note:

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor’s order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

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