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## EN

Braces in the hand area support damaged tissue structures (joints, capsules, ligaments, cartilages).

The semi-circular encompassing brace thereby supports the holding function of the ligaments and thus relieves the joint. The combined enclosure for the lower arm and wrist ensures that the brace cannot slip and stabilises the affected region. The semi-circular encompassing brace shell is shaped in such a manner that it holds the hand in a relaxed relieving posture. Furthermore, the brace also offers increased protection against incorrect movements.

### Instructions for use:

**Important:** Avoid temperatures in excess of 50 °C (hot water, sauna, vehicles) when using the brace.

The brace is anatomically pre-formed upon delivery. In order to optimise the fit, it can be heated to 90 °C (warming oven, customary baking oven) and shaped to match the patient’s characteristics. This should only be carried out by the specialist orthopaedic partner or by the physician. Once heated, the now flexible brace can be shaped directly on the skin after briefly being allowed to cool. The grey foam has an insulating effect. Close the fasteners and model the plastic shell to the patient’s arm after fitting the heated brace. The brace will cool down after several minutes and will retain the desired shape. Specific pressure points can be heated using a hairdryer and re-adjusted. The thermoplastic material allows this item to be re-adjusted over and over again during the treatment to retain an ideal fit.

The brace can be used in water without any problems whatsoever. Rinse with plenty of freshwater should the brace come into contact with saltwater. You can dry the back brace using a hair dryer on the lowest heat level. If you sweat excessively, take off the back brace at regular intervals, clean inside and outside and dry before putting it back on.

**Indications:** Pre-operative, post-operative, post-traumatic conditions, degenerative, e. g. carpal tunnel syndrome (CTS), tenosynovitis, peripheral nerve paralysis.

**Contraindications:** Lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

**Composition:** 40 % polyester, 40 % microfibre, 20 % polyamide

**Important Note:** No reutilisation – this medical device is meant for the care of one patient only. In case of doctor’s order to wear brace during night, avoid hindering blood circulation. In case of numbness loosen brace and remove it if necessary. If complaints continue, please contact the doctor.

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