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**EN**

Wrist supports relieve injured tissue structures (joint, ligaments, capsules, cartilages). The compression hereby supports the holding function of the ligaments and is thus relieving the joint.

The aluminium splint inlay is formed in a way that the hand is hold in a relaxed, easy position. Additionally the splint offers increased protection against wrong movements.

#### **Instructions for use:**

Bend aluminium splint to the desired form before putting it on. First close fastening between thumb and forefinger, then rest of Velcro fastenings.

#### **Indications:**

Chronic post-traumatic or post-operative irritations in the wrist area, i.e. Sprains, Carpal arthrosis, carpal tunnel syndrome (CTS), rheumatism, tenosynovitis.

#### **Contraindications:**

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

#### **Composition:**

68 % cotton, 15 % polyester, 12 % elastodies, 5 % polypropylene

#### **Important Note:**

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

The product contains latex and can trigger allergic reactions.

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