# BORT Daumen-Handorthese

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**REF** 103 500

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D10350012022-041003 ML I Rev.01

### DE BORT Daumen-Handorthese

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### EN BORT Thumb/Hand Brace

Many thanks for placing your trust in a medical device from BORT GmbH. Please read the existing instructions for use carefully. If you have any questions, please refer to your physician or the specialist retailer from whom you purchased this medical device.

#### Intended purpose

This medical device is a brace for immobilisation of the wrist and thumb in at least two directions of movement (ulnar/radial and dorsal/palm).

#### Indications

Pre-operative, post-operative, post-traumatic conditions, e.g. in case of carpal tunnel syndrome (CTS), stable carpal scaphoid fracture, stable radius fracture, ulnar ligament lesion (skier's thumb), tendinopathy, capsule strains.

#### Contraindications

Peripheral arterial occlusive disease (PAOD), lymph drainage disorders, also unclear soft tissue swellings distal to the aid positioned, sensory and circulatory disorders with the region of the body treated, skin diseases in the part of the body treated.

#### Application risks/Important notes

- This medical device is a prescribable product. Discuss the use and duration with your treating physician. The expert staff from whom you have received the medical device will select the appropriate size and instruct you regarding its use.
- remove the medical device prior to radiological examinations
  - in case of numbness, loosen the medical device and remove it if necessary
  - in case of persistent complaints, consult the physician or specialist retailer
  - use the medical device in accordance with therapeutic needs
  - only use other products simultaneously after consultation with your physician
  - do not make any changes to the product
  - do not wear it on open wounds
  - do not use in case of intolerance of one of the materials used
  - no re-use – this medical aid is intended for treating one patient
  - whilst wearing the brace, please neither use any creams nor ointments on or around it as they can destroy the material
  - do not heat in a microwave oven
  - please do not subject the medical device to direct heat or temperatures exceeding 50°C (e.g. hot water, vehicles, the sauna, heating) as this may result in damaging the material

#### Fitting instructions

**Important:** When delivered, the brace is anatomically pre-formed. To optimise the fit, it should be heated up to 90°C (using a heating oven or an ordinary commercial baking oven) and formed to fit the patient's circumstances. This should be done exclusively by specialist orthopaedic stores or the physician. The brace must not be heated in a microwave oven! Avoid temperatures exceeding 50°C (e.g. hot water, saunas, vehicles) during use. Open the brace before heating it. After being heated, the brace, which is flexible in the meantime, can be directly formed on the skin after cooling down briefly. The grey foam has an insulating effect. After fitting the heated brace, close the fasteners, starting at the metacarpophalangeal joint distally (on the metacarpus), then proximally (towards the elbow) and model the plastic shell on the patient's arm. Fastening straps which are too long can be shortened by cutting them off. Avoid entrapment of the skin. The brace cools down after a few minutes and retains the desired form. Isolated dents can be

heated and reworked using a hair-dryer. As thermoplastic material is used, this brace can repeatedly be refitted during the course of therapy in order to retain an optimum fit. The brace can be used in water without any problems. If it comes into contact with saline water, rinse it afterwards with ample fresh water. It is possible to dry the brace with a hair-dryer at the lowest heat level. In case of excess perspiration, remove the brace regularly, clean it inside and outside and dry it before putting it on. The understocking provided ensures extra wearing comfort. After putting the brace on, tighten the understocking towards the elbow and fingers in order to avoid the formation of creases under the brace.

#### Removal

To remove the brace, please open the Velcro fasteners and slip the brace off in the direction of the fingers.

#### Material composition

Polyamide (PA), polyester (PES), polyethylene (PE)  
The sewn in textile label provides the precise material composition.

#### Cleaning information

-  Hand-washing  Do not bleach  Do not dry in a tumble dryer
  -  Do not iron  Do not clean chemically
- Hand-wash the brace with lukewarm water and mild detergents. Air dry it.

#### Guarantee

The legal regulations of the country in which you acquired the product apply to the purchased product. Please contact your specialist retailer if you suspect a warranty claim. Please clean the product before submitting a warranty claim. If the enclosed instructions for use have not been properly observed, the warranty may be impaired or cancelled. The warranty does not cover use of the product inappropriate for the indication, non-observation of application risks, instructions and unauthorised modifications to the product.

#### Useful life/Lifetime of the product

The lifetime of the medical device is determined by its natural wear and tear if treated appropriately and as recommended.

#### Duty of notification

If a serious worsening of the state of health occurs when using the medical device, you are obliged to notify the specialist dealer or us as the manufacturer and the Medicines & Healthcare products Regulatory Agency (MHRA). You can find our contact information in these instructions for use. You can find the contact information for the national authority responsible for your country under the following link: [www.bort.com/md-eu-contact](http://www.bort.com/md-eu-contact).

#### Disposal

Upon the termination of use, the product must be disposed of in accordance with the corresponding local requirements.

#### Declaration of conformity

We confirm that this device conforms with the requirements of REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL. You can find the current declaration of conformity under the following link: [www.bort.com/conformity](http://www.bort.com/conformity)

Status: 04.2022

Medical device |  Single patient – multiple use

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