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## EN

Braces in the hand region support damaged tissue structures (joints, capsules, ligaments and cartilage).

Here, the brace construction supports the retaining function of the ligaments, thus relieving the joint. The combined thumb and metacarpophalangeal joint frame hinders the brace from slipping and stabilises the region concerned. The brace shell is shaped to retain the hand in a relaxed, relieving posture. Moreover, the brace provides increased protection against incorrect movement.

### Instructions for use:

**Important:** When delivered, the brace is anatomically pre-formed. To optimise the fit, it should be heated up to 90°C (using a heating oven or an ordinary commercial baking oven) and formed to fit the patient’s circumstances. This should be done exclusively by specialist orthopaedic stores or the doctor. The brace may not be heated in the microwave! Avoid temperatures exceeding 50°C (e.g. hot water, saunas, vehicles) during use.

Open the brace before heating it.

After being heated, the brace, which is flexible in the meantime, can be directly formed on the skin after cooling down briefly. The grey foam has an insulating effect. After fitting the heated brace, close the fasteners, starting at the metacarpophalangeal joint distally (on the metacarpus), then proximally (towards the elbow) and model the plastic shell on the patient’s arm. Fastening straps which are too long can be shortened by cutting them off.

Avoid entrapment of the skin. The brace cools down after a few minutes and retains the desired form. Isolated dents can be heated and reworked using a hair-dryer.

As thermoplastic material is used, this brace can repeatedly be refitted during the course of therapy in order to retain an optimum fit.

The brace can be used in water without any problems. If it comes into contact with saline water, rinse it afterwards with ample fresh water.

It is possible to dry the brace with a hair-dryer at the lowest heat level. In case of excess perspiration, remove the brace regularly , clean it inside and outside and dry it before putting it on.

The understocking provided ensures extra wearing comfort. After putting the brace on, tighten the understocking towards the elbow and fingers in order to avoid the formation of creases under the brace.

### Indications:

Pre-operative, post-operative, post-traumatic conditions, i.e. in the case of carpal tunnel syndrome (CTS), stable carpal scaphoid fracture, stable radius fracture, ulnar ligament lesion (skier’s thumb), tendinopathy, capsule strains, tendopathy, capsular ligament tears.

### Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

### Composition:

40 % polyamide, 30 % polyester, 30 % polyethylene

### Important Note:

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor’s order to wear brace during night, avoid hindering blood circulation. In case of numbness loosen brace and remove if necessary. If complaints continue, please contact the doctor.

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