IT

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EN

Immobilisation is a basic principle during treatment of injuries and overstress, as the recovery process is considerably accelerated. This splint is made of Polyester. By warming it to about 60 °C in warm water or with a hair-dryer its shape can be changed and thus altered at any time. After having been worn for some time, the splint can be adapted to possible anatomic changes. This should be done by an orthopaedic specialist or a doctor.

Important: Avoid temperatures over 40 °C (hot water, sauna, vehicles).

Indications:

Pre-operative, post-operative, post-traumatic conditions, carpal tunnel syndrome (CTS), paralysis, inflammatory and rheumatic diseases.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

100 % polyester

Important Note:

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear brace during night, avoid hindering blood circulation. In case of numbness loosen brace and remove it if necessary. If complaints continue, please contact the doctor.

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