IT

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**EN**

The Umbilical Hernia Support provides support in case of abdominal tissue infirmity in umbilical area. Supporting splints in dorsal area enhance the wearing comfort and avoid curling-up. The pad serves for re-positioning (pushing back) of the hernia. For this purpose the pad is reinforced by an inserted plastic core, and can be freely positioned on the entire support thanks to the hook strap.

#### **Instructions for use:**

Position pad onto the support by Velcro fixing and place onto the hernia port. Put support around the body and close it.

#### **Indications:**

Umbilical hernia, reducible umbilical hernias.

#### **Contraindications:**

Lymphatic drainage disorders including unclear soft tissue swelling, sensory disturbances and circulatory disorders in the specific body region, skin diseases in the specific part of the body.

#### **Composition:**

55 % polyester, 35 % polyamide, 10 % spandex

#### **Important Note:**

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

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