BORT Stoma-Protector

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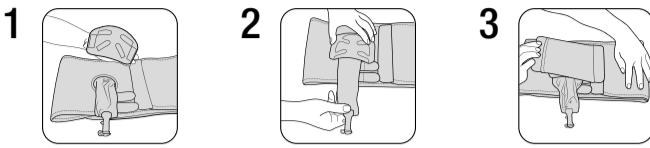
REF 104 099

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D10409912023-10I004 ML I Rev.01

DE BORT Stoma-Protector

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EN BORT Stoma Protector

Many thanks for placing your trust in a product from BORT GmbH. Please read the existing instructions for use carefully. If you have any questions, please refer to your physician or the specialist retailer from whom you purchased this product.

Intended purpose

This product is a protector which protects the stoma against pressure.

Application risks/Important notes

The expert staff from whom you have received the stoma protector will instruct you regarding its use.

- remove the stoma protector prior to radiological examinations
- only use other products simultaneously after consultation with your physician
- do not make any changes to the product
- do not use in case of intolerance of one of the materials used
- no re-use. This product is intended for treating one patient only

Fitting instructions

The BORT Stoma Protector protects the stoma against pressure (e.g. a safety belt when driving) and/or unintended physical influences (e.g. during sports activities). It can easily be attached to BORT stoma support (REF 104 090) using Velcro.

1 The BORT Stoma protector is attached to the front side of the BORT Stoma support using Velcro.

2 Position the protector above the stoma recess. Ensure that the stoma bag is not hemmed in by the protector. The stoma bag should sit loosely beneath the protector, enabling the bag to continue to fill unproblematically.

3 Next, pull the textile protector cover with the desired tension over the BORT Stoma protector and fix it simply using the integrated Velcro fastener on the stoma support.

Removal

To remove the protector, please take the textile cover off and remove the protector cautiously.

Material composition

Stoma protector: 100% polypropylene
Textile cover: 100% polyamide

Cleaning information

Hand-washing 30 °C Do not bleach Do not dry in a tumble dryer Do not iron Do not clean chemically
Before washing the BORT Stoma support, the textile protector cover and the BORT Stoma protector must be removed from the support. It can be hand-washed using mild detergents.

Guarantee

The legal regulations of the country in which you acquired the product apply to the purchased product. Please contact your specialist retailer if you suspect a warranty claim. Please clean the product before submitting a warranty claim. If the enclosed instructions for use have not been properly observed, the warranty may be impaired or cancelled. The warranty does not cover use of the product inappropriate for the indication, non-observance of application risks, instructions and unauthorised modifications to the product.

Useful life/Lifetime of the product

The lifetime of the product is determined by its natural wear and tear if treated appropriately and as recommended.

Disposal

Upon the termination of use, the product must be disposed of in accordance with the corresponding local requirements.

Declaration of conformity

We confirm that this device conforms with the requirements of REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL. You can find the current declaration of conformity under the following link: www.bort.com/conformity

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