IT

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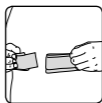
EN**Instructions for use:**

1. Application:
Slip the orthosis (elastic corset) onto the shoulders as if it were a backpack.

2. Velcro fastening/securing:
Bring the two ends of the straps forward around the chest. Then use the Velcro to fasten the straps together below the chest.

3. Fastening:
Then fasten both shoulder straps together beneath the chest using Velcro.

Indications: Straightening and relief of the thoracic spine, i.e. misalignment of thoracic spine, Back pain (pain in the region of the thoracic spine).

Contraindications: Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition: 41 % cotton, 36 % polyester, 17 % polyamide, 6 % polypropylene

Important Note: No reutilisation – this medical device is meant for the care of one patient only.
In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove if necessary. If complaints continue, please contact the doctor.
While wearing the support do not use creams or ointments. This can damage the material.

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