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EN

Back support for pregnant women, with lateral spring splints and straps for stabilisation and relief in the back and abdominal area. The back, abdominal and pelvic areas are relieved and supported; it exerts a pain reducing effect. Patient mobilisation is supported.

Instruction for use:

1. The brace should be applied while standing up.
2. Open all the Velcro straps and place the support around the hips.
3. Close the support across the abdomen.
4. At this point, pull the lateral straps to the front and fasten them so that noticeable relief in the back and abdominal area can be felt.

Composition:

60 % polyamide, 14 % spandex, 10 % polyester, 11 % PUR foam, 5 % viscose

Indications:

For stabilization of lumbar spine with metered pressure during pregnancy, for the treatment of pain in the lumbar area.

Contraindications:

Lymphatic drainage disorders including unclear soft tissue swelling, sensory disturbances and circulatory disorders in the specific body region, skin diseases in the specific part of the body.

Important Note:

No re-use – this medical aid is intended for treating one patient. If the support is to be worn at night, observe that the circulation is not impeded. In case of feelings of numbness, loosen the support and remove it if necessary. In case of persistent complaints, you should consult a doctor. Neither use any creams nor ointments when wearing the support as they could damage the material.

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