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EN

The BORT Abdominal Support helps in case of diminished connective tissue issue in the abdominal area.

Especially in the last third of pregnancy this weakness is increased due to specific hormones (Gestagens).

Instructions for use:

First place the wide abdominal band around the body below the stomach and close the right body side.

Pulling evenly on the two support bands attached at the sides, secure them to the front centre of the wide abdominal band. Abdominal and support bands should be fitted tightly enough to provide comfortable support while at the same time avoiding stases.

The four support rods can be removed from the rod pockets if and when required.

Indications:

Lumbar back pain in pregnancy, obesity, multiple pregnancy.

Contraindications:

Lymphatic drainage disorders including unclear soft tissue swelling, sensory disturbances and circulatory disorders in the specific body region, skin diseases in the specific part of the body.

Composition:

62 % polyamide, 24 % polyester, 12 % spandex, 2 % viscose

Important Note:

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

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