IT

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EN

Back Supports support injured tissue structures (bones, ligaments and cartilages). The compression hereby supports the holding function of the ligaments and is thus relieving the vertebrae. Fit and height are adapted to the female anatomy. The pad provides a massage effect which reduces swelling tendencies or which helps to an earlier restrain of swelling.

Instruction for use:

Place the support around the waist and – using the hand straps – evenly overlap the two Velcro fastening ends by tensioning slightly. Ensure that the support is centered and that the lower edge of the support is in the groin area (forward bend). This is important for optimal effect of the BORT StabuloBasic. The pad is velcroed to the support and can be positioned freely. Ensure that the wide side of the pad faces upward.

Indications:

Pain in the lumbar spine, recurrent low back pain, osteochondrosis, spondylarthrosis, spondylolysis without spondylolisthesis, mild lumbar deformity (degenerative spondylolisthesis), dorsalgia.

Contraindications:

Lymphatic drainage disorders including unclear soft tissue swelling, sensory disturbances and circulatory disorders in the specific body region, skin diseases in the specific part of the body.

Composition:

55 % spandex, 45 % polyamide

Important Note:

No re-use – this medical aid is intended for treating one patient. If the support is to be worn at night, observe that the circulation is not impeded. In case of feelings of numbness, loosen the support and remove it if necessary. In case of persistent complaints, you should consult a doctor. Neither use any creams nor ointments when wearing the support as they could damage the material.

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