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Back Supports support injured tissue structures. The compression of the support thereby supports the stabilisation function of the ligaments and thus relieves the vertebral spine. The integrated bridging segment, in connection with the circular enveloping support, causes a dislordosing as well as the avoidance of a hyperlordosing of the lumbar vertebra. The painful vertebral spine section will be "calmed down", the muscular system relieved and the recovery process stimulated. Due to the "module system", the stabilising effect of the support can be adapted to the course of therapy. Retrogression of the muscular system will be prevented by the proprioceptive effect of the BORT StabiloPlus Back Support.

Instructions for use:

On delivery the BORT StabiloPlus Back Support is pre-adjusted to maximal stabilisation. The bridging segment is fastened to the support by means of a textile pocket, in the splint pockets the stiffer version of the splints is provided. A more flexible splint set is attached in the packaging.

Put the support around the waist and by means of the hand loops fasten with a slight pull both Velcro closures evenly.

Pay attention that the support is placed central, and assure that the lower end of the support is ending in the inguinal region (trunk bending). Only this way an optimal effect of the BORT StabiloPlus Back Support can be achieved.

Tighten evenly first both lower straps; in the next step tighten the upper straps thus, that a comfortable fit of the BORT StabiloPlus is obtained. Too tight strapping may lead to drapery of the support.

For taking-off the support first loosen the straps, then open the support completely. During the course of therapy it might be necessary to reduce the stabilising effect of the BORT StabiloPlus Back Support. For this, remove the stiffer splint version from the attached splint pockets by folding back the reins and taking out the splints through the gap in the splint pocket. Now you can insert the short, more flexible splints.

Also the bridging segment can be removed from the textile pocket and being replaced by two splints. For this remove the textile pocket inclusive bridging element from the BORT StabiloPlus Back Support, take out of the textile pocket the bridging element through the gap at the rear and slip two of the long, more flexible splints into the provided receptors of the textile pocket. Subsequently put the textile pocket back into the BORT StabiloPlus by means of the Velcro fastener; the wide side faces thereby upwards.

Indications:

Moderate lumboischialgia with mild deficits with herniated discs/prolapse, moderate radicular, pseudo-radicular lumbar syndrome, grade I spondylolisthesis with low back pain, moderate lumbar deformity (recurrent treatment, complicated history), i.e. after inter-vertebral disc surgery with mild to moderate prolapse, Spinal stenosis, lumbar instability syndrome.

Contraindications:

Lymphatic drainage disorders including unclear soft tissue swelling, sensory disturbances and circulatory disorders in the specific body region, skin diseases in the specific part of the body, latex allergy.

Composition: 68 % elastodies, 32 % polyamide

Important Note:

No re-use – this medical aid is intended for treating one patient. If the support is to be worn at night, observe that the circulation is not impeded. In case of feelings of numbness, loosen the support and remove it if necessary. In case of persistent complaints, you should consult a doctor. Neither use any creams nor ointments when wearing the support as they could damage the material.

The product contains latex and can trigger allergic reactions.

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