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**EN**

The BORT Stabilo® Sacroiliac Joint Brace is a brace designed to stabilise and relieve the entire pelvic girdle and sacroiliac joints and to relieve the associated ligaments, cartilage and muscles.

By stabilising the pelvis, the BORT Stabilo® Sacroiliac Joint Brace helps reduce pain and improve mobility.

### Instruction for use:

Position the brace in the middle on the pelvis so that the upper edge of the brace is not higher than the upper iliac crest or protruding into the waist. If necessary, the position of the silicone pads can be adjusted individually, according to patient's needs, using the Velcro fastenings. Insert a finger in the finger loop provided on the fastener of the abdominal plate. Now pull the fastener evenly forwards. Press the left half of the fastener of the Stabilo® Sacroiliac Joint Brace onto the abdomen and place the right half of the fastener over the left half until you can attach it on the hook and loop pad on the left fastener half. When securing, withdraw the finger first from the left and then from the right finger loop. Now grasp both tension straps in both hands and pull them simultaneously and steadily forwards until the desired force effect (stabilisation) is produced.

Then secure both tension straps with the hook and loop fastener on the abdominal plate.

### Indications:

Sacroiliac joint arthrosis, sacroiliac joint instability, sacroiliac joint blockage, myalgias and tendopathies in the pelvic region, structural abnormalities after lumbar spinal spondylodeses.

### Contraindications:

Pain in the lumbar spine, recurrent low back pain, osteochondrosis, spondylarthrosis, spondylolysis without spondylolisthesis, mild lumbar deformity (degenerative spondylolisthesis), ischialgia, dorsalgia, latex allergy.

### Composition:

72 % polyamide, 22 % cotton, 6 % elastodies

### Important Note:

No re-use – this medical aid is intended for treating one patient. If the support is to be worn at night, observe that the circulation is not impeded. In case of feelings of numbness, loosen the support and remove it if necessary. In case of persistent complaints, you should consult a doctor. If pressure points, abrasions or similar occur, have your orthopaedic technician or doctor check the fit of the brace immediately. Neither use any creams nor ointments when wearing the support as they could damage the material.

The product contains latex and can trigger allergic reactions.

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