IT

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**EN**

Hernia Trusses have been a proven therapy concept for decades in case of resettable inguinal hernia (i.e. the hernia can be forced back). The pad closes the point of hernia and avoids thus emerging and incarceration of the hernia.

#### Instructions:

First position the pad onto the point of hernia. Then place the strap around the body in height of the pelvic (between waist and hip) and close it.

Finally bring the thigh belt between the legs around the thigh and close it with the metal buckle which is sewn onto the strap in the hip area.

The pads are fixed by Velcro closures and therefore they can be individually positioned. This allows optimal positioning onto the point of hernia.

#### Indication:

Repositionable, unilateral or double-sided inguinal hernia.

#### Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

#### Composition:

68 % viscose, 32 % elastodien

#### Important Note:

No reutilisation – this medical device is meant for the care of one patient only.

Wear support only when under strain, not during rest or sleep. Avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary.

If complaints continue, please contact the doctor.

The product contains latex and can trigger allergic reactions.

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