IT

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**EN**

Immobilisation is a basic principle during treatment of injuries and overstress, as the recovery process is considerably accelerated.

The perforation provides better airing and thus increases comfort for the wearer. The brace is water-resistant and easy care.

**Indications:**

Post-traumatic, post-operative conditions, i.e. after sprain, contusion, ligament lesions (skier's thumb), immobilisation in the case of rheumatic-inflammatory processes, rhizarthrosis.

**Contraindications:**

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

**Composition:**

95 % polypropylene, 5 % polyamide

**Important Note:**

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear brace during night, avoid hindering blood circulation. In case of numbness loosen brace and remove it if necessary. If complaints continue, please contact the doctor.

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