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EN BORT HerniaFix Elastic Spring Truss, Double-Sided

Many thanks for placing your trust in a medical device from BORT GmbH. Please read the existing instructions for use carefully. If you have any questions, please refer to your physician or the specialist retailer from whom you purchased this medical device.

Intended purpose

This medical device is a hernia truss which is suitable for reducing hernia of the abdominal wall (inguinal region).

Indications

Reducible inguinal hernia.

Contraindications

Unclear soft tissue swellings, sensory and circulatory disorders with the region of the body treated, skin diseases in the part of the body treated, non-reducible inguinal hernia, incarcerated hernia, latex allergy.

Application risks/Important notes

- This medical device is a prescribable product. Discuss the use and duration with your treating physician. The expert staff from whom you have received the medical device will select the appropriate size and instruct you regarding its use.
- Trial fitting and size determination possible with closed packaging! The goods can only be taken back if the hygienic packaging is undamaged
 - remove the medical device prior to radiological examinations
 - if wearing at night is prescribed, avoid negative influence on the circulatory system
 - in case of numbness, loosen the medical device and remove it if necessary
 - in case of persistent complaints, consult the physician or specialist retailer
 - use the medical device in accordance with therapeutic needs
 - only use other products simultaneously after consultation with your physician
 - do not make any changes to the product
 - do not wear it on open wounds
 - do not use in case of intolerance of one of the materials used
 - no re-use. This medical aid is intended for treating one patient only
 - whilst wearing the support, please neither use any creams nor ointments on or around it as they can destroy the material

Fitting instructions

Hernia trusses for reducible inguinal hernia (i.e. the hernia can be pushed back into its cavity) have comprised a proven therapy concept for several decades. The pad presses against the hernial ring, thus hindering the emergence and trapping of the hernia.

For attaching the aid, the patient should be placed on an appropriate examination table in psaos position: for this purpose, the patient lies flat on his back, the legs are elevated. The lower legs are placed at right angles to the thighs on a surface which should be as high as the length of the thighs. The inguinal hernia should then no longer be palpable, that is to say not emerge from the abdomen.

Then, position the pads on the hernial rings. Then wrap the truss (pelvic

restraint) around the body at the level of iliac crest (between the waist and hip) and fasten it.

If necessary, the spring steel springs integrated into the pelvic restraint can be adjusted to the patient.

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