

**HerniaFix® special hernia truss with
an adjustable round pad**

Art. no. 110 100

Hernia trusses have been a proven therapeutic concept for repositionable inguinal hernia (i.e. the hernia can be pressed back into place) for several decades. The pad closes the hernial ring, thus hindering the displacement and trapping of the fracture



Instructions for use

Place the truss in a recumbent position. Reposition the fracture manually and then place the pad on the hernial ring. Following this, lay the truss around the body at the height of the iliac crest (between the waist and the hip) and close it.

The pad can be turned 45° by loosening the two screws and moved 4cm vertically. This enables adaptation in different positions and ideal setting for the hernial ring. Finally, retighten the screws.

Care instructions

Hand wash the truss with lukewarm water and mild detergents. Air dry. Avoid heat.

Indications

Repositionable inguinal hernia

Contraindications

Non-repositionable inguinal hernia, AVK, CVI, incarcerated inguinal hernia, latex allergy


Important information

No re-use – this medical aid is intended for treating one patient.

If night-time wearing has been prescribed, avoid impairment of the blood circulation. In case of feelings of numbness, loosen the support and remove it if necessary. In case of persistent complaints, consult a doctor.

Iliac crest circumference sizes

- small 85 - 94 cm
- medium 95 - 105 cm
- large 106 - 115 cm

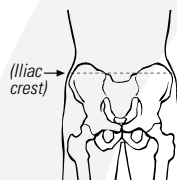

Material composition

40 % polyamide, 36 % polyester, 12 % elastodiene, 12% cotton



The product contains latex and may trigger allergic reactions

