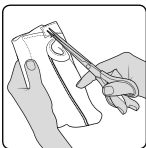
IT

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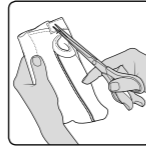
**EN**

Wrist Supports support injured tissue structures (joint, ligaments, capsules, cartilages). The compression hereby supports the holding function of the ligaments and is thus relieving the joint. At the same time the support helps to a high degree to avoid wrong movements.

This support supports the wrist and the back of the hand. Due to the thumb opening the support does not slip.

#### Instructions for use:

The thumb opening can be opened by unpicking the thread facing the thumb.

#### Indications:

Minor sprains, strains, after removal of plaster cast, tenosynovitis.

#### Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

**Composition:** 68 % polyamide, 25 % elastodien, 7 % viscose.

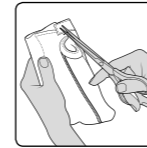
**Important Note:** No reutilisation – this medical device is meant for the care of one patient only. In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor. While wearing the support do not use creams or ointments. This can damage the material.

The product contains latex and can trigger allergic reactions.

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#### Indications :

Distorsion légère, claquage, post-plâtre, tendovaginite.

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