IT

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**EN**

Wrist Supports support injured tissue structures (joint, ligaments, capsules, cartilages). The compression hereby supports the holding function of the ligaments and is thus relieving the joint. Due to the thumb opening the support does not slip. The highly elastic form-knit allows easy support application and yet provides a good supporting effect.

#### **Instruction for use:**

Grasp the edge of the support and pull it over the hand. The opening for the thumb prevents the support from slipping

#### **Composition:**

68 % polyamide, 25 % elastodien, 7 % viscose

#### **Indications:**

Chronic, post-traumatic or post-operative irritations in the area of saddle joint of the thumb, i.e. in the case of ulnar and radial ligament instability, early stages of rhizarthrosis

#### **Contraindications:**

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

**Important Note:** No re-use – this medical aid is intended for treating one patient.

If the support is to be worn at night, observe that the circulation is not impeded. In case of feelings of numbness, loosen the support and remove it if necessary. In case of persistent complaints, you should consult a doctor.

Neither use any creams nor ointments when wearing the support as they could damage the material.

The product contains latex and can trigger allergic reactions.

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