IT

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EN

Immobilisation is a basic principle during treatment of injuries and overstress, as the recovery process is considerably accelerated.

Instruction for use:

Important: Avoid temperatures over 40 °C (hot water, sauna, vehicles).

The BORT **SellaFix® Thumb Braces** are made of thermoplastic material which has considerable advantages, since the brace can easily be moulded and individually adjusted:

- focal-point corrections can be made with a hot air gun.
- when dipped briefly in warm water (60 °C), the thumb piece can be corrected.
- the shape can easily be changed with the help of scissors.

The multiple possibilities of correction allow perfect fitting for the expert. The perforation provides better airing and thus increases comfort for the wearer. The brace is water-resistant and easy care. The padding can be individually positioned.

Indications:

Post-traumatic, post-operative conditions, i.e. after sprain, contusion, ligament lesions (skier's thumb), immobilisation in the case of rheumatic-inflammatory processes (rhizarthrosis).

Contraindications: Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

80 % polyester, 15 % polyamide, 5 % spandex

Important Note:

No reutilisation – this medical device is meant for the care of one patient only. In case of doctor's order to wear brace during night, avoid hindering blood circulation. In case of numbness loosen brace and remove it if necessary. If complaints continue, please contact the doctor.

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