IT

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**EN**

Immobilisation is a basic principle during treatment of injuries and overstress, as the recovery process is considerably accelerated.

Wrist Supports support injured tissue structures (joint, ligaments, capsules, cartilages). The compression hereby supports the holding function of the ligaments and is thus relieving the joint.

The aluminum splint inserted is shaped to hold the hand in a neutral position. Additionally the splint offers increased protection against wrong movements.

#### Instructions for use:

Before fitting, have the aluminum splint formed physiologically by your specialist supplier as required. First close fastening between thumb and forefinger, then rest of Velcro fastenings. Finally close the Velcro fastener at the last thumb joint.

#### Indications:

Pre-operative, post-traumatic and post-operative conditions, i.e. after thumb ligament reconstruction, immobilisation in the case of rheumatic-inflammatory processes, i.e. rhizarthrosis, in the case of degenerative diseases such as carpal tunnel syndrome.

#### Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

#### Composition:

86 % polyester, 14 % polyamide-silk

#### Important Note:

No reutilisation – this medical device is meant for the care of one patient only. In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

The product contains latex and can trigger allergic reactions.

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