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**EN**

Wrist Supports support injured tissue structures (joint, ligaments, capsules, cartilages). The compression hereby supports the holding function of the ligaments and is thus relieving the joint.

The thumb cut-out prevents the support from shifting. The elastic Stabilo® strap can be applied customized and the plastic splint offers maximum support. Due to the highly elastic formed knit, the support is easy to apply, yet provides good support.

#### Instructions for use:

Plastic splint adjustment: At a temperature of approx. 70 – 160 °C, the plastic splint material can be individually formed. After forming and before applying the support, please push the individually formed plastic splint into the splint pocket of the support.

#### Indications:

Chronic, post-traumatic or post-operative irritations in the thumb saddle joint area, i.e. late stage post-fracture, arthritis, ulnar and radial ligament lesions.

#### Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

#### Composition:

Knitted fabric: 68 % polyamide, 25 % elastodien, 7 % viscose

#### Important Note:

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove if necessary. If complaints continue, please contact the doctor.

While wearing the support do not use creams or ointments. This can damage the material.

The product contains latex and can trigger allergic reactions.

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