IT

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**EN**

Dynamic finger splints are applied for after-treatment of injuries/affections of extension resp. flexion ligament. They result in flexion/extension of the finger joint and thus relieve the effected finger. Finger splints are also used to prevent or treat contractions of the flexion or extension ligaments.

#### Instructions for use:

Position splint with first pad (to be recognized by the white coated spring wire) behind palm of hand at the base of the finger (A) and place the second pad after distal finger fold (B) – see illustration. The pressure can be increased or decreased by carefully bending the spring.

#### Indications:

Post-operative, post-traumatic extension of PIP joints.

#### Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

#### Composition:

Pad: 100 % EVA  
Wire frame: spring steel type 1.4310

#### Important Note:

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear splint during night, avoid hindering blood circulation. In case of numbness loosen splint and remove it if necessary. If complaints continue, please contact the doctor.

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