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EN

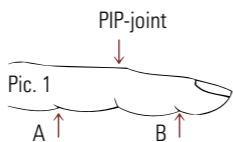
BORT Finger Splints are used to normalize function of the ligaments. By means of the dynamic extension/flexion the effected ligament sections are relieved or calculated trained.

Instructions for use:

The finger splint can be used for the PIP joint (see picture 1) as well as for the DIP joint (see picture 2). Pressure can be increased or decreased by carefully bending the spring.


Application as PIP-splint:

Position splint between base of the finger (A) and distal finger fold (B) – see picture 1.

Application as DIP-splint:

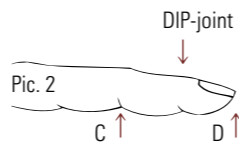Position splint between proximal finger fold (C) and distal finger end (D) – see picture 2.

Indications:

Post-operative, post-traumatic flexion of PIP (L1) or DIP (L2) joints.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

Pad: 100 % PE foam 125/EVA
Wire frame: spring steel type 1.4310

Important Note:

No reutilisation – this medical device is meant for the care of one patient only. Avoid hindering blood circulation. In case of numbness loosen bandage and remove it if necessary. If complaints continue, please contact the doctor.

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