IT

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EN

Back Supports support injured tissue structures (bones, ligaments and cartilages). Hereby the compression effect supports the holding function of the local ligament and muscular function, and thus relieves the vertebral column. The specifically developed nap pad stimulates by its intermittent masage effect the intercellular metabolism by an increased blood circulation.

Instructions for use:

Place the back support in a way that the wider part of the pad is heading upwards. At the same time the pad should be oriented in a way that the naps are not placed in the middle of the vertebral column, rather do „massage“ the lateral areas. Close the back support for the back at abdomen area by means of Velcro closure. The support should tightly fit, nowever not cramp. The compression of the pad can be adjusted by additional elastic straps, which also are to be closed by Velcro at abdominal side.

Indications: Treatment of increased pain in the lumbar spine, severe low back pain, recurrent low back pain, osteochondrosis, spondylarthrosis, mild lumboischialgia, mild radicular syndrome, spondylolysis without spondylolisthesis, mild to moderate lumbar deformity (degenerative spondylolisthesis), after inter-vertebral disc surgery.

Contraindications:

Lymphatic drainage disorders including unclear soft tissue swelling, sensory disturbances and circulatory disorders in the specific body region, skin diseases in the specific part of the body, latex allergy.

Composition:

84 % polyester, 16 % polyamide

Important Note:

No reutilisation – this medical device is meant for the care of one patient only. In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

The product contains latex and can trigger allergic reactions.

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