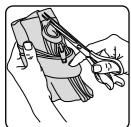
IT

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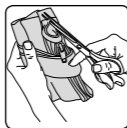
EN

Wrist Supports support injured tissue structures (joint, ligaments, capsules, cartilages). The compression hereby supports the holding function of the ligaments and is thus relieving the joint.

Due to the thumb opening the support cannot slip.

The soft elastic Stabilo® band can be individually adjusted and the mouldable aluminium splint gives maximum support. By means of the zip the support can easily be applied providing nevertheless good support.

Instructions for use:

Bend aluminium splint to the desired form before putting it on.

The thumb opening can be opened by unpicking the thread facing the thumb.

The circular Stabilo® strap has two differently coloured Velcro surfaces. Always fasten the strap with the transparent area on the fastening point of the brace.

Indications: Chronic, post-traumatic or post-operative irritations in the wrist area, i.e. Sprains, Carpal arthrosis

Contraindications: Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

Composition: 68 % polyamide, 25 % elastodies, 7 % viscose

Important Note: No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

While wearing the support do not use creams or ointments. This can damage the material.

The product contains latex and can trigger allergic reactions.

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