IT

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EN

Back Supports support injured tissue structures (bones, ligaments and cartilages). Hereby the compression effect supports the holding function of the local ligament and muscular function, and thus relieves the vertebral column. Integrated stabilising elements provide additional protection against wrong movements.

Instructions for use:

Close the back support for the back at abdomen area by means of Velcro closure. Use the finger loops for comfortable fit. The support should tightly fit, nowever not cramp. The pad can be variably positioned in the lumbar region of the support. The wide part of the pad must face up towards the head.

Indications:

Treatment of pain in the lumbar spine, low back pain, recurrent low back pain, osteochondrosis, spondylarthrosis, spondylolysis without spondylolisthesis, mild lumbar deformity (degenerative spondylolisthesis).

Contraindications:

Lymphatic drainage disorders including unclear soft tissue swelling, sensory disturbances and circulatory disorders in the specific body region, skin diseases in the specific part of the body.

Composition:

36 % nylon, 24 % helanca, 20 % spun rayon, 20 % elasthan

Important Note:

No reutilisation – this medical device is meant for the care of one patient only. In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor. While wearing the support do not use creams or ointments. This can damage the material.

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