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EN

Back Supports support injured tissue structures (bones, ligaments and cartilages). Hereby the compression effect supports the holding function of the local ligament and muscular function, and thus relieves the vertebral column. The specifically developed nap pad stimulates by its intermittent massage effect the intercellular metabolism by an increased blood circulation.

Instructions for use:

Wrap the back brace around your body and fasten the two-piece Velcro strap. Patients with larger bodies are advised to fasten the Velcro strap in the middle of the trunk and then shift the brace to the left counterclockwise until the elastic body part is in the middle. Check the position of the brace and correct as appropriate. Then attach the additional two elastic straps with the desired tightness.

Indications:

Treatment of increased pain in the lumbar spine, severe low back pain, recurrent low back pain, osteochondrosis, spondylarthrosis, mild lumboischialgia, mild radicular syndrome, spondylolysis without spondylolisthesis, mild to moderate lumbar deformity (degenerative spondylolisthesis), after intervertebral disc surgery.

Contraindications:

Lymphatic drainage disorders including unclear soft tissue swelling, sensory disturbances and circulatory disorders in the specific body region, skin diseases in the specific part of the body, latex allergy.

Composition:

50 % cotton, 35 % viscose, 15 % elastodies

Important Note:

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

The product contains latex and can trigger allergic reactions.

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