IT

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**EN**

Immobilisation is a basic principle during treatment of injuries and overstress, as the recovery process is considerably accelerated. The splint is softly padded, thus allowing comfortably fitting. The integrated anatomically preformed plastic splint guarantees optimal fit.

#### Indications:

Post-operative, post-traumatic, degenerative, inflammatory conditions, i.e. in the case of arthritis, ulnar/radial lesion of thumb ligaments, late stage following tendon reconstruction, rhizarthrosis.

#### Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

#### Composition:

86 % polyester, 14 % polyamide-silk

#### Important Note:

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear splint during night, avoid hindering blood circulation. In case of numbness loosen splint and remove it if necessary. If complaints continue, please contact the doctor.

The product contains latex and can trigger allergic reactions.

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