IT

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**EN**

Wrist Braces support injured tissue structures (joint, ligaments, capsules, cartilages). The compression hereby supports the holding function of the ligaments and is thus relieving the joint.

The aluminium splint inlay is formed in a way that the hand is hold in a relaxed, easy position. Additionally the splint offers increased protection against wrong movements.

#### Instructions for use:

Before putting on, bend the aluminium splints to the desired shape. Position the dorsal splint in one of the two available splint bags, dependent on the indication position. First close the fastener between the thumb and index finger, then the following Velcro fasteners.

#### Indications:

Pre-operative, post-operative, post-traumatic, e. g. sprain, degenerative, e. g. carpal tunnel syndrome (CTS), tenosynovitis, peripheral nerve paralysis.

#### Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

#### Composition:

Knitted fabric: 84 % polyester, 13 % polyamide-silk, 3 % elastodies

#### Important Note:

No reutilisation – this medical device is meant for the care of one patient only. In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

The product contains latex and can trigger allergic reactions.

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