IT

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**EN**

Wrist Supports support injured tissue structures (joint, ligaments, capsules, cartilages).

The aluminium splint inlay is formed in a way that the hand is held in a relaxed, easy position. Additionally the splint offers increased protection against wrong movements.

**Instructions for use:**

Before attaching the splint, open all Velcro straps. Put splint onto of the hand and the inside of the arm. Fix Velcro closures in the following sequence: wrist, forearm, back of the hand. Cut off redundant Velcro closures. Integrated aluminium splint can be manually adjusted.

**Indications:**

Post-operative, post-traumatic conditions, carpal tunnel syndrome (CTS), sprain.

**Contraindications:**

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

**Composition:**

38 % polyamide-velour, 28 % cotton, 28 % polyurethane, 6 % polyester

**Important Note:**

No reutilisation – this medical device is meant for the care of one patient only. In case of doctor's order to wear splint during night, avoid hindering blood circulation. In case of numbness loosen splint and remove it if necessary. If complaints continue, please contact the doctor.

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