IT

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EN

Immobilisation is a basic principle during treatment of injuries and over-stress, as the recovery process is considerably accelerated.

Instructions for use:

Prior to putting-on the splint open all Velcro straps. Put splint on the back of the hand and fasten Velcro closures around wrist and forearm. Finally fasten the hand bracket with the Velcro strap. Excess length of the Velcro straps may be cut-off.
Palmar bracket and integrated alu-splint to be manually adjusted.

Indications:

Post-operative, post-traumatic conditions, carpal tunnel syndrome (CTS), sprain.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

52 % polyurethane, 25 % polyamide, 15 % polyamide velour, 8 % polyester

Important Note:

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear splint during night, avoid hindering blood circulation. In case of numbness loosen splint and remove if necessary. If complaints continue, please contact the doctor.

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