IT

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**EN**

Immobilisation is a basic principle during treatment of injuries and overstress, as the recovery process is considerably accelerated. The DigiSoft® Finger Brace is made of a specially soft, skin-friendly material. The inserted alu-splint can be adjusted individually.

**Instructions for use:**

Bend to required shape before fitting the aluminium splint. First open all Velcro tapes, then place the affected finger(s) on the brace. Then fasten the Velcro tapes in the finger region such that the fingers are comfortably supported. Close the Velcro tape at the wrist and wrap the long, loose, enclosed tape around the middle of the hand and fasten.

**Indications:** Post-operative, post-traumatic conditions, i.e. in the event of correction, positioning, immobilisation of the interphalangeal joint.

**Contraindications:** Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

**Composition:**

74 % polyamide, 26 % polyurethane

**Important Note:**

No reutilisation – this medical device is meant for the care of one patient only. In case of doctor's order to wear brace during night, avoid hindering blood circulation. In case of numbness loosen brace and remove it if necessary. If complaints continue, please contact the doctor.

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