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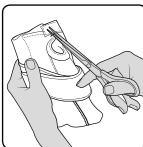
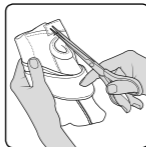
EN

Wrist Supports support injured tissue structures (joints, ligaments, capsules, cartilages). The compression supports the stabilization function of the ligaments and thus relieves the joint.

Due to the thumb recess the support cannot slip. The semi-elastic Stabilo® strap can be put on individually dosed, at the same time it serves for fixing of the silicone pad, which can be freely positioned, for therapy-supporting compression of a ganglion. The deformable alu-splint offers optimal stability.

Instructions for use:

Adaptation of the anatomically pre-shaped plastic splint: If required, the material of the plastic splint can be individually adapted by the orthopaedic technician at a temperature of about 180 °C. Allow the reshaped splint to cool and then re-insert into the splint pocket of the support.

Prior to putting-on bend the plastic splint into the desired form.

The thumb opening can be opened by unpicking the thread facing the thumb. Put support on and fix the Stabilo® strap with the non-elastic part in the area of the volar plastic splint. Depending on the location of ganglion fasten the silicone pad to the Stabilo® strap Velcro section. Subsequently put the Stabilo® strap circular around the support, thus that the silicone pad is placed between support and Stabilo® strap. Direct contact of the silicone pad has to be avoided!

Indications: Chronic post-traumatic or post-operative irritations in the wrist area, i.e. Sprains, Carpal arthrosis, intracapsular ganglion.

Contraindications: Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

Composition: 68 % polyamide, 25 % elastodien, 7 % viscose

Important Note:

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove if necessary. If complaints continue, please contact the doctor.

While wearing the support do not use creams or ointments. This can damage the material.

The product contains latex and can trigger allergic reactions.

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